

# QUICK REFERENCE GUIDE

## ELEGANZA 2

Hospital bed for acute care





### SUPERVISOR CONTROL PANEL


### HANDSET

### SIDE RAIL

### FUNCTION LOCK

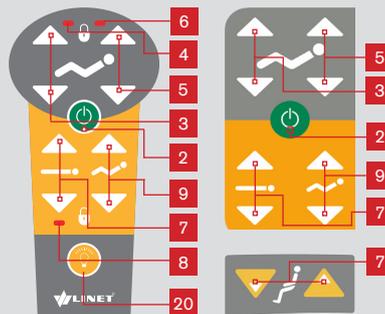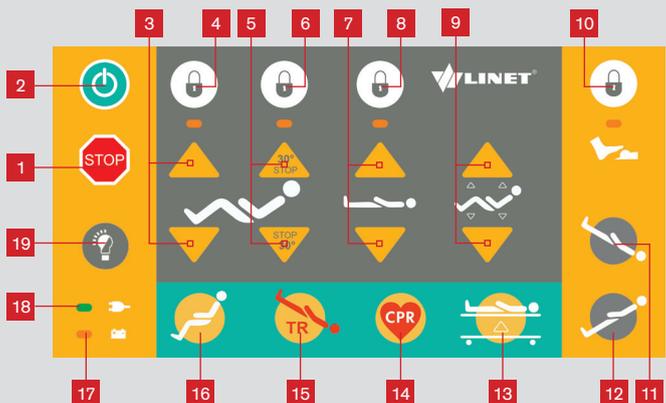

### Satellite panel

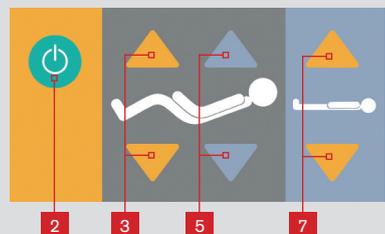

- |                            |                                      |
|----------------------------|--------------------------------------|
| 1. Central STOP            | 11. Trendelenburg position           |
| 2. Activating GO           | 12. Reverse Trendelenburg position   |
| 3. High rest adjustment    | 13. Examination position             |
| 4. Thigh rest lock         | 14. CPR (Resuscitation)              |
| 5. Backrest adjustment     | 15. Emergency Trendelenburg position |
| 6. Backrest lock           | 16. Cardiac chair position           |
| 7. Height adjustment       | 17. Battery charge status indicator  |
| 8. Height adjustment lock  | 18. Mains power indicator            |
| 9. Auto-Contour adjustment | 19. Bed Lights control               |
| 10. Foot Control lock      | 20. Flashlight                       |

### DISABLING FUNCTIONS

1. Activate the keypad (on supervisor control panel) by pressing the GO button (2)
  2. Press the respective function lock button (4, 6, 8 or 10)
- Respective lock indicator light is illuminated and function is locked.

### ENABLING FUNCTIONS

1. Activate the keypad (on supervisor control panel) by pressing the GO button (2)
  2. Press the respective lock button (4, 6, 8 or 10)
- Respective lock indicator light is no longer illuminated and function is unlocked.

LINET UK, LINET House 17 Murrills Estate,  
Portchester, Hampshire, PO 16 9RD  
Tel.: 0844 248 1833, Fax: 0844 499 4409  
e-mail: info@linet.uk.com, [www.linet.uk.com](http://www.linet.uk.com)


# QUICK REFERENCE GUIDE

## ELEGANZA 2

Hospital bed for acute care



### BED DESCRIPTION

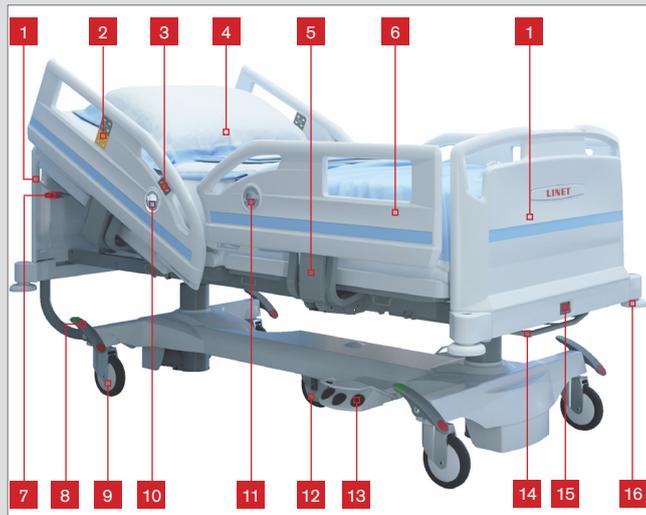

- |   |                                       |
|---|---------------------------------------|
| 1. Head and foot end                      | 9. Castor                             |
| 2. Integrated control panel in side rails | 10. Angle indicator - illuminated     |
| 3. Integrated Mobilift controller         | 11. Angle indicator - non illuminated |
| 4. Mattress platform                      | 12. Fifth castor                      |
| 5. Side rail release mechanism            | 13. Foot controller                   |
| 6. Split plastic side rails               | 14. Linen/Supervisor panel holder     |
| 7. CPR control lever                      | 15. Bed end - locking mechanism       |
| 8. Castor control lever                   | 16. Protective bumper                 |

### CENTRAL BRAKING




- Forward movement**  
5th castor determines direction
- Unrestricted movement**  
All castors are unbraked
- Braked**  
All castors are braked

### PLUG & PLAY



**Handset**  
To connect handset to the bed, simply install it in to the plug and play connector located near the accessory rails. Disable the handset by removing it from the plug and play slot.

### SIDE RAIL OPERATION

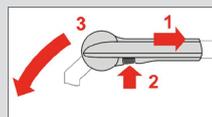

- 3/4 side rails**
- Grip the release mechanism
  - Push the side rail backwards 1
  - Push the locking button 2
  - Fold the side rails down 3



- Split plastic side rails**
- Grip the side rail.
- Push the side rail toward the middle of the bed 1
  - Pull the locking lever 2
  - Side rail will fold down smoothly 3

### CPR



- Push the CPR (red) lever up.
- Push the backrest down using its handle.

### ANGLE INDICATOR



**Head side rail**  
The angle indicator in head side rail is illuminated. A green light indicates the lowest position of bed. A white light indicates all other positions.



**Foot side rail**  
The angle indicator in the foot side rail is not illuminated and is used for angle measurement of Trendelenburg and Reverse Trendelenburg position.

### BED END LOCK



LOCKED



UNLOCKED

